

Technical Sheet

REV. 07 - 15/05/2020 - Mod. 71A5 - EN

PBS 5



GENERAL DATA

Disposable container for sharp and cutting objects disposal. Item designed for low volumes of waste (care tray, trolley, domestic care, doctor's office)

- ▶ **Material:** Plastic material 100 % virgin Polypropylene
- ▶ **Colour:** Yellow body and red lid in order to signal the dangerous content.
Materials coloured with masterbatch without heavy metals and cadmium. Materials compliant to REACH
- ▶ **Shape:** Truncated cone
- ▶ **Weight:** 365 gr.
- ▶ **Max Load:** 2.400 gr.
- ▶ **Nominal Capacity:** 5,000 Lt.
- ▶ **External dimensions:** Ø 214 x 195h mm (body and lid assembled)
- ▶ **Base dimensions:** Ø 182 mm



LOGISTIC DATA:

- ▶ **Pcs per carton:** 55 (body and lid not assembled)
- ▶ **Carton dimensions:** 79x53x49 H cm
- ▶ **Pcs per pallet:** 440 pcs
- ▶ **Pallet dimensions:** 120x80x212 h cm

CERTIFICATIONS:

- ▶ Compliant to AFNOR NF X-30-511
- ▶ Compliant to NF Mark
- ▶ Compliant to ADR - UN Approved
- ▶ Compliant to ISO 23907-1 Kite Mark
- ▶ Compliant to FDA K071517
- ▶ Compliant to TRBA 250

INSTRUCTIONS

► Lid assembly:

Firmly press on all the perimeter, until the lid clicks on the body



► Temporary closure

The container could be temporarily closed using the cap upside down.

At this stage, closure is temporary



USE:

► During use, close the cap in temporary closure position



IMPORTANT!

Before use, press on the anti-reflux fins in order to create a funnel-shaped opening. This device eases the insertion of the waste and creates a barrier to prevent from leakage



► Permanent closure

The container could be closed permanently applying pressure on the cap until it clicks on the lid. At this stage, closure is permanent



► Close the container permanently once full




- Large opening to ease the disposal of scalpels, test tubes, etc
- Autoclave Suitable to autoclave at 134° for 18 minutes
- Once filled, the containers have to be disposed through incineration, releasing no toxic emissions

► Needles removal:

Use the “V” removal device to separate needle and syringe


► Insulin pens removal:

Use the “marguerite” device to unscrew the insulin pen

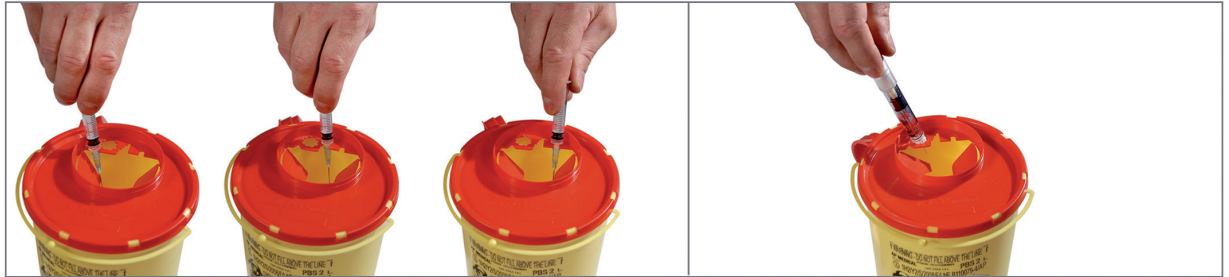

AVAILABLE HOLDERS:

- Trolley/wall bracket

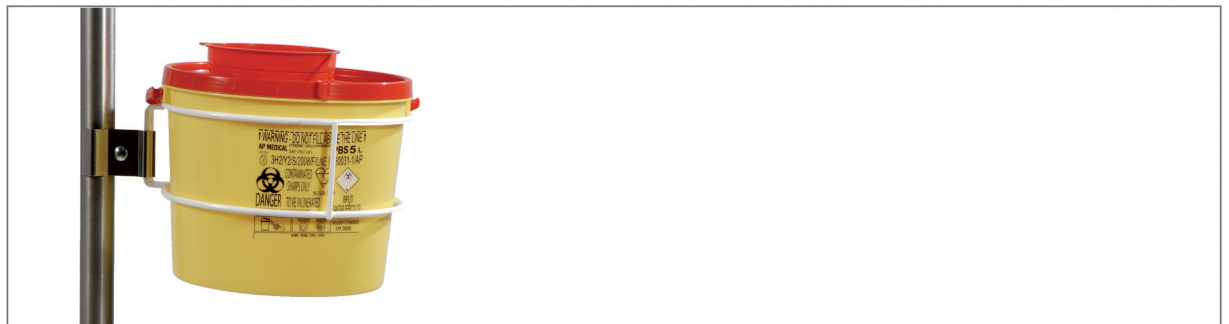

ADVANTAGES:

- Lid provided with safe closure device, permanent closure, prevents from waste reflux
- Easy to use and to handle
- Sharp and cutting objects resistant
- Handles available
- Lid provided with needles removal devices
- Suitable to autoclave (compliant to NFX 30-511 even after sterilization)
- Maximum filling level indicated
- Several holders available
- Irreversible permanent closure
- To be disposed through incineration, releasing no toxic emissions

SAFETY INFORMATION:

- Do not use the product before fitting the lid
- Do not attempt to rescue objects already in the container
- Respect the filling level indicated
- Use the container only with appropriate lid
- Do not try to open the container once permanently and irreversibly sealed
- Follow the instructions of use to ensure the safety of the container
- Container to dispose according to the existing law
- Use handles to move the container